



Suite One, 77 The Fleet, Belper,
Derbyshire. DE56 1NW. UK
Tel/Fax: 01773 820056
pdml2003@yahoo.com
www.peaksanddalesmedical.co.uk

Peaks & Dales Medical Ltd

Serving the ever changing needs of Healthcare

Important information on how to use Dispo-Bano patient washing glove

Please read carefully

DO NOT HOLD GLOVE UNDER A RUNNING TAP-THIS WILL WASH AWAY ALL OF THE PRE DRIED CLEANSER

DO NOT SUBMERGE THE GLOVE AND SQUEEZE IT CONTINUOUSLY-THIS WILL ALSO WASH AWAY ALL OF THE PRE-DRIED CLEANSER

Just moisten it by adding a small amount of warm water (5 mls).

Squeeze gently until the cleansing gel becomes activated.

Then add more warm water as required.

Once the glove becomes foamy wash the patient.

When you have washed the patient **DO NOT RINSE** - just towel dry.

Dispo-Bano is not soap and leaves no residue.

Dispo-Bano will not sting the eyes- it has a "no tears" formulation

Discard the glove after each patient use.

Dispo-Bano glove is single use only.

If you require further help and advice about using **Dispo-Bano Gloves** correctly, Please call Customer Support who will be delighted to assist you.

They can be contacted on **07850 099326**