



Suite One, 77 The Fleet, Belper,
Derbyshire. DE56 1NW. UK
Tel/Fax: +44 1773 820056
pdml2003@yahoo.com
www.peaksanddalesmedical.co.uk

Peaks & Dales Medical Ltd

Serving the ever changing needs of Healthcare

Important information on how to use

Dispo-Foam *patient cleansing sponges*

Please read carefully

DO NOT HOLD SPONGE UNDER A RUNNING TAP-THIS WILL WASH AWAY ALL OF THE PRE DRIED CLEANSER

DO NOT SUBMERGE THE SPONGE AND SQUEEZE IT CONTINUOUSLY-THIS WILL ALSO WASH AWAY ALL OF THE PRE-DRIED CLEANSER

How To Use

Just remove the sponge from the wrapper and **moisten** it by adding a small amount of warm water.

Squeeze the sponge vigorously until the cleanser becomes activated.

Then add more warm water and keep squeezing..

Once the sponge becomes foamy wash the patient.

After washing **DO NOT RINSE**-just towel dry.

Dispo-Foam is not soap and leaves no residue.

Dispo-Foam will not sting the eyes- it has a "no more tears" formulation

Dispo-Foam is also shampoo and Conditioner.

Discard the sponge after each patient use.

Dispo-Foam is single use only.

If you require further help and advice about using Dispo-Foam correctly, Please call **Customer Support** who will be delighted to assist you.

They can be contacted on **01773 820056** or **07850 099326**